

Vinoxx by maxill provides low level hard surface disinfection and is designed for use in dental and medical clinics, hospitals, nurseries, nursing homes and schools where disinfection and prevention of cross-contamination is of prime importance. This product is safe for use on non-porous environmental hard and washable surfaces, such as counters, trays, tables, walls, etc. Vinoxx by maxill is safe for use on non-porous, washable, non-critical medical devices such as bedpans, dental chairs, surgical light handles, etc. This product will not stain or leave grit or soap scum. Non-abrasive Vinoxx by maxill is formulated for use in daily maintenance programs with a balance of detergents, disinfectants, and malodour ingredients that deliver effective cleaning, disinfecting and malodour control.

#### PRODUCT INFORMATION

Vinoxx by maxill will **sanitize** (provides a 99.9% reduction) hard and non-porous non-food contact surfaces according to the method for Sanitizers -Non Food Contact Surfaces, modified in the presence of 5% blood serum, in 30 seconds against:

*Enterobacter aerogenes*  
*Staphylococcus aureus* (Staph)  
*Salmonella choleraesuis* (Salmonella)  
*Streptococcus pyogenes* (Strep)  
*Escherichia coli* 0157:H7 (pathogenic strain) (E. coli)  
*Enterococcus faecalis* (Vancomycin resistant) (VRE)

Vinoxx by maxill provides **complete bactericidal activity** against:

*Pseudomonas aeruginosa*  
*Salmonella choleraesuis*  
*Staphylococcus aureus*

\*Vinoxx by maxill provides **complete virucidal activity** against:

Human Immunodeficiency Virus type 1 (HIV-1)\*\*  
Influenza A/Hong Kong (Influenza) virus

#### DIRECTIONS FOR USE

Vinoxx by maxill is ready to use. Do not add water or other chemicals to the container. Test product on an inconspicuous area first to ensure the disinfectant is compatible with the surface (e.g. no discoloration, etc.)  
Pre-clean the surface by applying this product and wiping to remove organic and other soils. To disinfect the pre-cleaned surface, apply Vinoxx by maxill liberally until the surface is thoroughly wet. Ensure the surface remains wet for at least 1 minute at Room Temperature. Then wipe the surface dry.



#### Cleaning Care for Dental Chairs

In addition to disinfecting, a cleaning protocol is required by the chair manufacturer. Call 1-800-268-8633 or visit [www.maxill.com/ca/chaircare](http://www.maxill.com/ca/chaircare) for more information.

DIN 02244220

ITEM 60004

# Vinoxx<sup>TM</sup>

by maxill

## HARD SURFACE DISINFECTANT & GERMICIDAL CLEANER

FAST ACTING, CLEANS & DEODORIZES

#### CAUTION

- Eye irritant
- Keep out of reach of children
- For external use only

Read the Label Before Using  
For Institutional Use Only

946 mL

#### ACTIVE INGREDIENTS

Alkyl (40% C<sub>12</sub>, 50% C<sub>14</sub>, 10% C<sub>16</sub>) dimethylbenzyl ammonium chlorides.....0.3%

**BACTERICIDAL**

**VIRUCIDAL\***

(when used as directed)

**\*\*This product is intended for use against HIV-1 in settings where the virus would be expected to be encountered, such as settings where contamination by blood or body fluid is likely.**

**Contact Time/Concentration:** Vinoxx is effective against HIV-1 in the presence of 5% blood serum with a 30 second contact time.

**Personal Protection:** Personnel that clean items soiled with blood or body fluids must wear appropriate personal protective equipment (i.e., barrier protection), such as disposable gloves, gowns, masks or eye coverings.

**Cleaning:** Target surfaces must be cleaned prior to disinfection, because the presence of heavy soil on a surface may reduce the intended disinfectant efficacy of the product.

**Infectious Materials Disposal:** Any materials used in the cleaning process that may contain blood or body fluids are to be disposed of immediately in accordance with local regulations for infectious materials disposal.

Refer to [Directions for Use](#).

#### PRECAUTIONS:

**CAUTION:** Causes eye irritation. Avoid contact with eyes, skin or clothing. Avoid contact with foods.

#### FIRST AID:

**IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

**IF ON SKIN OR CLOTHING:** Take off contaminated clothing. Rinse skin immediately with plenty of water for 15 – 20 minutes. Call a Poison Control Center or doctor for treatment advice. Have the product container or label with you when calling a Poison Control Center or doctor or going for treatment.

#### NOTE TO PHYSICIAN:

Probable mucosal damage may contraindicate the use of gastric lavage.

#### STORAGE:

Store closed in original container at Controlled Room Temperature.

#### DISPOSAL:

Rinse the empty container. Do not reuse the empty container for alternate uses. Dispose of the container in accordance with municipal and provincial requirements. For information on spills and for clean up of spills or the disposal of unused, unwanted product, consult the SDS or contact maxill or the Provincial Regulatory Agency.

**maxill inc.**

St. Thomas, ON, Canada, N5R 6C8  
[www.maxill.com](http://www.maxill.com) 1-800-268-8633



Made in Canada from domestic and imported ingredients

REV.2020/07