

## Product Description

tb Minuteman is an intermediate level hard surface disinfectant as well as a tuberculocide, virucide and fungicide. tb Minuteman is designed for use in dental and medical clinics, hospitals and nursing homes where disinfection and prevention of cross-contamination is of prime importance. tb Minuteman is safe for use on non-porous environmental hard surfaces, such as counters, trays, tables, etc. tb Minuteman will not stain or leave residual smears after drying. tb Minuteman is not for use in hospital nurseries.

## Product Information

tb Minuteman provides complete tuberculocidal activity against:

- *Mycobacterium bovis* (BCG)

tb Minuteman provides complete bactericidal activity against

- *Pseudomonas aeruginosa*
- *Salmonella choleraesuis*
- *Staphylococcus aureus*
- Methicillin Resistant *Staphylococcus aureus* (MRSA)
- Vancomycin Resistant *Enterococcus faecalis* (VRE)

tb Minuteman provides complete virucidal activity against:

- *Herpes simplex virus type 1*
- *Herpes simplex virus type 2*
- *Human immunodeficiency virus type 1* (HIV-1)\*
- *Influenza A virus*
- Poliovirus type 1, Sabin

tb Minuteman provides complete fungicidal activity against:

- *Trichophyton mentagrophytes*

## To Pull Towelettes through the Lid,

1. Unscrew entire lid from the canister.
2. Remove the protective seal from the canister.
3. Pull up a corner of the first towelette from the centre of the roll and push 2 cm through the X in the lid.
4. Replace lid tightly on the canister.
5. Lift the flip-top and pull out the towelettes one at a time.

## Directions for Use

Pre-clean the surface by wiping with a towelette to remove organic and other soils. Dispose of towelette. To disinfect the pre-cleaned surface, apply tb Minuteman liberally, by wiping the surface with a towelette until the surface is thoroughly wet. Ensure the surface remains wet for at least 1 minute at Room Temperature. Then wipe the surface dry.

Once opened, the unused portion of the product may be stored in the original container for up to 60 days until used.

maxill

DIN 02282461



Fast Acting

TUBERCULOCIDE/BACTERICIDE/VIRUCIDE/FUNGICIDE

**Minuteman**

**HARD SURFACE DISINFECTANT**

READY TO USE TOWELETTES • CLEANS AND DEODORIZES

**THE 1 MINUTE TUBERCULOCIDE, BACTERICIDE  
VIRUCIDE & FUNGICIDE FOR WASHABLE HARD SURFACES**

- Flammable
- Store away from heat sources
- Do not expose to an open flame
- Keep out of reach of children
- For external use only

## Active Ingredients

Ethyl Alcohol.....	79.0% w/w
Benzalkonium Chloride.....	0.2% w/w
o-Phenylphenol.....	0.1% w/w

**Unscented**  
**ITEM 61163U**

**160**  
**TOWELETTES**



6 0 7 0 5 6 1 1 6 3 U

Made in Canada with domestic and imported parts

## Precautions:

May cause eye irritation. Avoid contact with eyes and skin. Use in a well ventilated area. Avoid excessive inhalation of vapours. Keep out of reach of children.

**First Aid:** If solution is swallowed, give a glass of water. If contact with eyes or skin occurs, flush thoroughly with water. For excessive inhalation, remove patient to fresh air. Get medical attention if ingested, if contact with eyes occurs or if an irritation persists.

**Storage:** Towelettes are saturated with flammable liquid. Keep away from heat, sparks and open flames. Store closed in original container at Room Temperature.

**Disposal:** Rinse the empty container. Do not reuse the empty container for alternate uses. Dispose of the container in accordance with municipal and provincial requirements. For information on spills and for clean up of spills or the disposal of unused, unwanted product, consult the SDS or contact maxill or the Provincial Regulatory Agency.

**\*This product is intended for use against HIV-1 in settings where the virus would be expected to be encountered, such as settings where contamination by blood or body fluid is likely.**

**Personal Protection:** Personnel that clean items soiled with blood or body fluids must wear appropriate personal protective equipment (i.e., barrier protection), such as disposable gloves, gowns, masks or eye coverings

**Cleaning:** Target surfaces must be cleaned prior to disinfection, because the presence of heavy soil on a surface may reduce the intended disinfectant efficacy of the product.

**Infectious Materials Disposal:** Any materials used in the cleaning process that may contain blood or body fluids are to be disposed of immediately in accordance with local regulations for infectious materials disposal.

Refer to **Directions for Use**.

**maxill**  
St. Thomas, ON, Canada, N5R 6C8

[www.maxill.com](http://www.maxill.com) 1-800-268-8633

**INSTITUTIONAL USE ONLY • READ THE LABEL BEFORE USING**

REV.2017/06